PROTECT & DEFEND

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SPRAY NASALE con lattoferrina

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MEDICAL DEVICE

NASAL SPRAY with lactoferrin

Indications: the medical device PROTECT & DEFEND is a hypertonic solution, with Lactoferrin, with a decongestant action. Used for washing the nasal cavities, it creates a barrier that protects the nasal mucosa from the entry of viruses and bacteria. The barrier effect is assisted by the action of lactoferrin, which by binding iron prevents the proliferation of viruses and bacteria. Useful for relieving the symptoms of stuffy nose facilitates the removal of allergens and exogenous microbial agents, thus reducing local inflammation and infections of the nasal cavities and upper airways and contributing to the improvement of the mucociliary function and the relief of symptoms.

Composition: Lactoferrin, sodium chloride, hydroxypropylmethylcellulose (HPMC), potassium dihydrogen phosphate, potassium sorbate, water.

Directions: shake before use. On first use or after a prolonged period of inactivity, operate the dispenser several times until the first dispensing of the product.

From 6 to 12 years: 1 spray per nostril, 2 times a day.

From 12 years up: 2 sprays per nostril, 2 times a day.

Remove the protective cap from the dispenser. Gently insert the dispenser tip into the nostril, holding the bottle vertical.

Dispense one actuation for each nostril by pressing the pump quickly. After use, clean the dispenser and put the protective cap back on it.

Warnings: external use. Do not swallow. Carefully close the package after use. Keep out of reach of children. Do not use in case of hypersensitivity or allergy to one of the components. Lactoferrin is a protein extracted from milk and therefore it is not recommended for use in case of allergy to milk proteins.

There are no known interactions with active pharmaceutical ingredients and in the absence of clinical data available for use in pregnancy, lactation, under 6 years and in case of concomitant use with drugs, it is advisable to consult a doctor. Prolonged use of topical products can give rise to sensitization phenomena: if such phenomena occur, it is advisable to suspend the treatment and consult a doctor. Do not use if package is damaged. Do not use beyond the expiration date. If symptoms persist, consult a doctor. Once the container is opened, the medical device can be used within 28 days.

Storage: store in a cool and dry place, away from heat sources. The expiry date refers to the product correctly stored in an unopened package.

Content: 20 ml e
0.68 fl.oz.

Last revision date: April 2021