PROTECT & DEFEND

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SPRAY OCCHI
con lattoferrina e acido ialuronico

SPRAY AD OCCHI CHIUSI

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MEDICAL DEVICE

OCULAR SPRAY

with lactoferrin and ialuronic acid

EYE CLOSED SPRAY

Indications: The medical device PROTECT & DEFEND is indicated to relieve the symptoms of irritation and dry eyes. By preventing the dehydration of the ocular surface altered by environmental stress (air conditioning, exposure to wind, dust, allergens, polluted air) and visual stress (prolonged use of contact lenses, video screens, unfavourable light conditions) it helps to maintain the natural defences of the eye. It gives relief to the eye and the periorcular and eyelid area in the case of sensitive, dry and irritated eyes, even when wearing contact lenses. Based on hyaluronic acid and Lactoferrin. High molecular weight hyaluronic acid lubricates and hydrates the eye surface. Lactoferrin is a substance normally present in the tear fluid which, by modulating the presence of iron, helps to inhibit microbial growth.

Lactoferrin accounts for about 25% of tear proteins. Aging and some eye diseases such as dry eye, keratitis and conjunctivitis of different etiologies cause a decrease in lacrimal lactoferrin, making these individuals more susceptible to eye infections. The formulation with lactoferrin helps to maintain the natural defences of the eye when reduced by ocular diseases and also acts as an antioxidant acting as a barrier for free radicals.

Composition: Lactoferrin, hyaluronic acid, sodium chloride, phenoxyethanol, vitamin E, polysorbate-20, water.

Directions: Remove the protection cap. Shake before use. Keep at a distance of 10 cm from the closed eyelid. Spray 1-2 times on the closed eyelid 3-4 times a day. Suitable for daily use.

Warnings: external use. Do not spray in the open eye. Do not swallow. Carefully close the package after use. Keep out of reach of children. Do not use in case of hypersensitivity or allergy to one of the components. Lactoferrin is a protein extracted from milk and therefore it is not recommended for use in case of allergy to milk proteins.

There are no known interactions with active pharmaceutical ingredients and in the absence of clinical data available for use in pregnancy, lactation, under 6 years and in case of concomitant use with drugs, it is advisable to consult a doctor. Prolonged use of topical products can give rise to sensitization phenomena: in the event that such phenomena should occur, it is advisable to suspend the treatment and consult a doctor. Do not use if package is damaged. Do not use beyond the expiration date. If symptoms persist, consult a doctor. Once the container is opened, the medical device can be used for up to 6 months.

Storage: store in a cool and dry place, away from heat sources. The expiry date refers to the product correctly stored in an unopened package.

Content: 10 ml e
0.34 fl.oz.

Date of last revision April 2021

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**Eurochem Ricerche S.r.l.**
Gialean S.r.l. Via Amman, 19 - 33084 Cordenons (PN) - ITALY