PROTECT & DEFEND

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Contenuto: 10 ml e
0.34 fl.oz.

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PROTECT & DEFEND

NOSE AND LIPS OINTMENT with lactoferrin

MEDICAL DEVICE

Indications: The medical device PROTECT & DEFEND is an ointment for the nose and lips with a soothing, emollient and protective action. Indicated for the area around the nose and lips when it is irritated or dry due to colds, rhinitis, wind, cold and the continuous use of handkerchiefs. The barrier effect is assisted by the action of lactoferrin, which by binding iron protects the mucous membranes from the entry of viruses, bacteria and other exogenous agents. Almond oil has a lubricating and emollient action that relieves dry and irritated skin.

Composition: Lactoferrin, karité butter, almond oil, vitis oil, vegetable wax, beeswax, ester of behenic acid, vitamin E, potassium sorbate.

Directions:

- Wash your hands before using the product.
- Spread a generous layer of ointment on the lips and around the nostrils and massage gently.
- Close well after use.
- Apply several times a day as needed

Warnings: external use. Do not swallow. Carefully close the package after use. Keep out of reach of children. Do not use in case of hypersensitivity or allergy to one of the components. Lactoferrin is a protein extracted from milk and therefore it is not recommended for use in case of allergy to milk proteins. Do not use on children under the age of 6. If you are pregnant or breastfeeding, consult your doctor before use.

There are no known interactions with active pharmaceutical ingredients and in the absence of clinical data available for use in pregnancy, lactation, under 6 years and in case of concomitant use with drugs, it is advisable to consult a doctor. Prolonged use of topical products can give rise to sensitization phenomena: in the event that such phenomena should occur, it is advisable to suspend the treatment and consult a doctor. Do not use if package is damaged. Do not use beyond the expiration date. If symptoms persist, consult a doctor. Once the container is opened, the medical device can be used within 90 days.

Storage: store in a cool and dry place, away from heat sources. The expiry date refers to the product correctly stored in an unopened package.

Content: 10 ml e
0.34 fl.oz.

Date of last revision: April 2021